

MAZIC D Material
Compatibility Table_V6



	Material										Material									
	Machine Name	TEMP	SG	Mode 2.0	Ortho Model	Cast	Splint	Denture	Etc.		Machine Name	TEMP	SG	Model 2.0	Ortho Model	Cast	Splint	Denture	Etc.	
DLP	MAZIC D P500	●	●	●	●	●			Machine Setting	SLA	Sindoh A1+Plus	Laser Power :28 Default	Laser Power :32 Default	Laser Power :25 Default	Laser Power :25 Default					
	Hunter/p100	●	●	●	●	●					Formlab 2	Denture Teeth A2 50um								
	Carima 1M2	●	●	●	●	●					Zenith U	Dreve_Castable_SoftVAT	Clear_SG_SoftVAT	Dreve_Castable_SoftVAT		Castable Red_SoftVAT				
	Carima 1M0(1M3)	●	●	●	●	●			Machine Setting	LCD	Karv1. 2	●	●	●	●	●				
	Vetz 30 D2	●	●	●	●	●					Ackuretta	●	●	●	●	●				
	Asiga-MAX UV	●	●	●	●	●					Freeshape 120	●	●	●	●	●				
	Zenith D	●	●			●					Phrozen	●	●	●						
	BI03D W11	Nextdent SG & Ortho	Nextdent SG & Ortho	Nextdent Model B : 80s O : 8s	Nextdent SG & Ortho	Nextdent SG & Ortho					Soinc Mini	●	●	●						
	envisionTEC E1	●	●								Sonic 2K	●	●	●	●	●				
	Nextdent 5100	Nextdent MFH & Ortho Rigid	Nextdent SG	Nextdent Model 2.0 Gray	Nextdent Ortho Rigid						XL 4K	●	●	●	●	●				
	Kulzer cara 4.0	dima Impression Blue 100um	dima guide 100um	dima Print Model HD Standard 50um	dima guide 100um						Shuffle XL	●	●	●	●	●				
											Shuffle K. Edition		15-16s /100um	●						
											Shuffle 2K	●	15-16s /100um							
										Anycubic Photon Zero	About 20-25s /50um	About 10-12s /50um	●	●	About 10-12s /50um					
									Photon	●	●	●		●						
									Photon S	●	●	●		●						
									Photon Mono X	50um B : 40s O : 7s	50um B : 35s O : 6s									
									Uniz Slash 2	●	●	●								
									Shash+		●	●	●							
									Dazz L120 Zeron3D Pixel one	●	●	●								
									WANHAO Duplicator D7		●	●								



Available



In Progress



Double click & Download



Closed Type